


Fitting of Alpha Junior Anti-entrapment cover

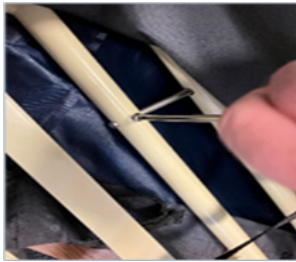

Opening the covers

Open the bag carefully ensuring that no sharp instruments come in contact with the covers.

Slacken the two mattress retainer screws and remove bar.



Replace bar.

Replace bar through the eyelets of the cover and tighten the two screws.



Cover

Place cover over the head end as image and lower down flat.

Make sure you do not trap any of the plastic connectors on the sheets, with the bed frame.



Attaching the cover

Remove the thumb screws, place cover over the threaded holes and re fit the thumb screw. Do this all down one side of the bed. (Note: it is easy to do this when the platform is on its side and not flat). Attach the Velcro at the ends of the sheet to the platform cross bar.



Straps

Attach both sides of the connectors and adjust the strap to fit snug (do not pull tight). Make sure all movements of the bed can be completed without damaging the cover.



Finish

The cover should look like this and check that the bed profiles fully without impedance before signing off.

NOTE: Only to be fitted by a trained person.